

ONE-CARE®

INFANT SAFETY LANCETS



Heel blood sampling has never been easier and safer...

One-handed operation for ease of use ...

Unique lock that prevents premature firing: simply SLIDE the lock to the side and PRESS trigger button to activate

Sterile, single use design with activation indicator for safety ...

Welded housing and permanent blade retraction prevents possible re-use or injury, depressed trigger button provides activation indication to distinguish between used and non-used device

Ergonomic shape for convenience ...

It enables different ways of holding, while ribbed design prevents slippage from a hand during use

Mother with child image for positive feeling, because we care ...

It reflects the heart of ONE-CARE® device with inside mechanism designed for gentle incision

Visual Guides

for precise placement on the recommended test site



INSTRUCTIONS FOR USE

■ Newborn screening has never been easier and safer ...



STEP 1

REMOVE ONE-CARE™ heel lancest from blister pack, **CLEAN** the test site



STEP 2

UNLOCK lancest by sliding the cherry lock button to the side



STEP 3

PLACE lancest on the desired site of baby's heel*

NOTE*



STEP 4

PRESS the trigger button to activate lancest



STEP 5

DISPOSE lancest in sharps container as per your facility's safety procedures



NOTE*

The marked areas of a foot indicate the recommended sites for testing.

COLOR-CODED VERSIONS FOR EASE OF CHOICE

■ Because every baby is unique ...

	 MICRO PREMIE YELLOW	 PREMIE PURPLE	 NEWBORN GREEN	 TODDLER BLUE
DESCRIPTION	MICRO PREMIE YELLOW	PREMIE PURPLE	NEWBORN GREEN	TODDLER BLUE
INCISION (DEPTH /WIDTH)	0.65 mm / 1.4 mm	0.85 mm / 1.75 mm	1.00 mm / 2.50 mm	2.00 mm / 3.00 mm
INCISION (DEPTH /WIDTH)	Ref. 1401 50 pcs/box 10 boxes/case	Ref. 1402 50 pcs/box 10 boxes/case	Ref. 1403 50 pcs/box 10 boxes/case	Ref. 1404 50 pcs/box 10 boxes/case

WWW.ONE-CARE.COM

