

ONE-CARE[®]

INSULIN SYRINGES



Insulin delivery has never been comfortable...

Convenient
Wide flaps for convenient finger recess



Efficient
Thin-Wall Technology for optimal insulin flow

Gentle
Precise needle grinding and special lubrication for gentle injection



Easy
Smooth move of the plunger

Optimal
Flat-base design for injection at the right 90-degree angle and low impact on the skin

Precise
Precise graduation and marking for accurate dosing


Feature / Benefit

Low Dead Space to minimize insulin loss


High "Flow Rate"

Correct injection angle of 90-degree and low impact on the injection site


Traditional design

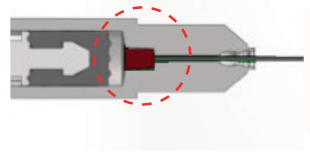

High dead space, cone design (red) between syringe barrel and needle

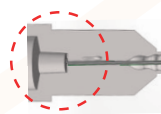


Misalignment between canula needle and syringe barrel. Not centered needle reduces the inner diameter and results in greater flow resistance during injection

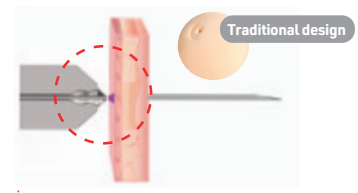

Cone design of the syringe barrel results in different injection angles and high impact on the injection site


MediVena ONE-CARE[®] New Design

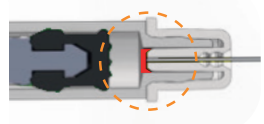

Low dead space, reduction of cone area (red) between syringe barrel and needle



Precise needle orientation for optimal insulin flow

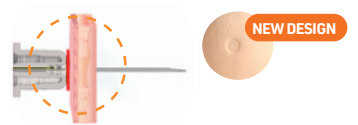

Flat base provides stable skin contact platform and promotes injection at the correct angle of 90-degree with low impact on the skin



ONE-CARE[®]


Insulin Syringes

MORE INFORMATION
READ THE QR-CODE




INSTRUCTIONS FOR USE


- Intuitive and easy to follow steps for user comfort.

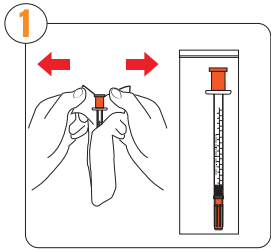


Open the package, remove the device.

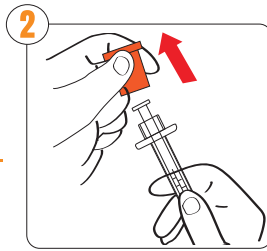


Remove the plunger cap.

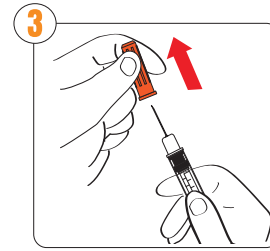


Remove the needle safety cap.

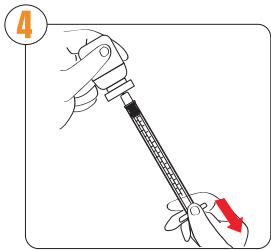

Prepare medication according to the facility's protocol.

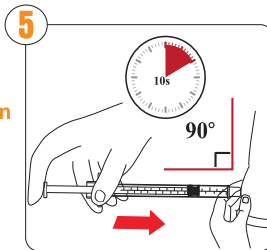

Perform injection and deliver dose. Wait 10 seconds after depressing the plunger to ensure that the entire dose is delivered.

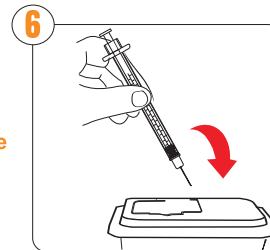

Dispose of the used device into a sharps container.

ORDERING INFORMATION

REF	DESCRIPTION	PACKAGING
2231	ONE-CARE® Insulin Safety Syringes 0.3cc, 31G x 5/16" (8mm)	100 pcs /box 12 bx /case
2232	ONE-CARE® Insulin Safety Syringes 0.5cc, 31G x 5/16" (8mm)	100 pcs /box 12 bx /case
2233	ONE-CARE® Insulin Safety Syringes 1cc, 31G x 5/16" (8mm)	100 pcs /box 12 bx /case
2234	ONE-CARE® Insulin Safety Syringes 0.3cc, 30G x 5/16" (8mm)	100 pcs /box 12 bx /case
2235	ONE-CARE® Insulin Safety Syringes 0.5cc, 30G x 5/16" (8mm)	100 pcs /box 12 bx /case
2236	ONE-CARE® Insulin Safety Syringes 1cc, 30G x 5/16" (8mm)	100 pcs /box 12 bx /case
2237	ONE-CARE® Insulin Safety Syringes 1cc, 30G x 1/2" (12.7mm)	100 pcs /box 12 bx /case
2238	ONE-CARE® Insulin Safety Syringes 1cc, 29G x 1/2" (12.7mm)	100 pcs /box 12 bx /case
2239	ONE-CARE® Insulin Safety Syringes 1cc, 28G x 1/2" (12.7mm)	100 pcs /box 12 bx /case
2240	ONE-CARE® Insulin Safety Syringes 1cc, 27G x 1/2" (12.7mm)	100 pcs /box 12 bx /case



WWW.ONE-CARE.COM

