

ONE-CARE[®]

SAFETY LANCETS **OPTI**



Blood micro-sampling has never been easier...

Easy

Long safety tab for easy twist off and pull out



Comfortable

Dual spring safety activation mechanism for enhanced patient's comfort

Ergonomic

Ergonomic T-shape body for comfortable grip

SAFE

- All needles are gamma-sterilized and protected with a sterility tab
- Fully enclosed housing ensures that needle is hidden before and after use
- Single-use mechanism prevents sharps injuries and reuse of the device

COMFORTABLE

- Silicone-coated, ultra-thin needles ideally positioned during skin penetration with double-spring mechanism to reduce vibrations
- Color-coded product versions to meet specific blood sample requirements and address a variety of patient skin types
- Ergonomic design for comfortable grip and secure blood collection

SIMPLE

- Easy to use, intuitive operation in 3 steps only, no pre-loading required
- Long safety tab designed for easy removal - simply twist off the tab and pull it out
- Optimal skin contact area for precise positioning on the desired test site and adequate blood flow


Every product detail designed with ease of use and comfort in mind




INSTRUCTIONS FOR USE

Blood micro-sampling has never been easier ...


- Intuitive design with only 3 simple steps.

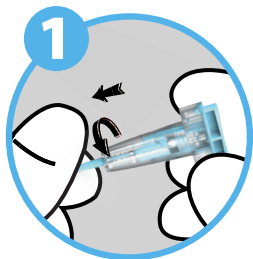

1 TWIST OFF and PULL OUT the safety cap

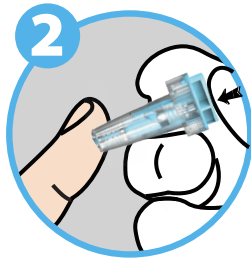

2 Place ONE-CARE® PRO lancet on the test site and press the top push button to activate the device

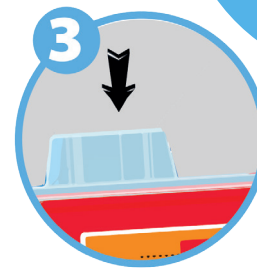

3 DISPOSE of lancet into sharps container

TEST SITE PREPARATION AND SELECTION

Test Sites

Prick the side of the finger, not the pad



- Patient should wash hands with warm water and soap before blood collection
- Test site should be disinfected with alcohol swab
- Blood collection should be performed on the sides of the fingertip, not the pads
- Index, middle and ring fingers are recommended
- Apply intermittent pressure on the finger below test site to enhance blood flow if needed

PRODUCT SELECTION CHART

Available in several color-coded versions to meet different test blood flow requirements and address a variety of patient skin types.



BOX 100 PCS

REF	DIAMETER (G) / DEPTH (mm/in)	COLOR	PACKAGING
1041	30G / 0.7 mm	Purple	100 pcs / box
1042	28G / 1.0 mm	Sky Blue	100 pcs / box
1043	25G / 1.1 mm	Dark Blue	100 pcs / box
1044	21G / 1.8 mm	Green	100 pcs / box

BOX 200 PCS

1031	30G / 1.5 mm	Light Green	200 pcs / box 18 bx (3,600 pcs) / case
1032	28G / 1.6 mm	Light Blue	200 pcs / box 18 bx (3,600 pcs) / case
1033	25G / 1.6 mm	Dark Pink	200 pcs / box 18 bx (3,600 pcs) / case
1034	23G / 1.8 mm	Blue	200 pcs / box 18 bx (3,600 pcs) / case
1035	21G / 1.8 mm	Dark Green	200 pcs / box 18 bx (3,600 pcs) / case
1036	21G / 2.4 mm	Green	200 pcs / box 18 bx (3,600 pcs) / case
1037	21G / 2.8 mm	Pink	200 pcs / box 18 bx (3,600 pcs) / case
1038	17G BLADE / 1.0 mm	Yellow	200 pcs / box 18 bx (3,600 pcs) / case
1039	17G BLADE / 1.5 mm	Orange	200 pcs / box 18 bx (3,600 pcs) / case
1040	17G BLADE / 2.0 mm	Red	200 pcs / box 18 bx (3,600 pcs) / case